



How to tighten GEN 2 Wheelchair wheels

If wheels are crooked, wobbly or unstable, then the wheel needs to be tightened.

Written By: Kali Slaughter



INTRODUCTION

This guide shows how to tighten the main wheels on the wheelchair.

TOOLS:

- [Socket 19mm](#) (1)
 - [19 mm wrench](#) (1)
-

Step 1 — How to tighten GEN 2 Wheelchair wheels



- Inspect chair to see which wheel is loose.
- Tighten the outer bolt with a 19 mm socket wrench.
- Tighten the nut on the inside with a 19 mm wrench.

If wheel does not feel properly secured, keep tightening the wheel in small amounts until the wheel is secure.

This document was last generated on 2017-12-21 02:40:50 AM.