



How to Tighten GEN 2 Wheelchair Wheels

If GEN 2 wheelchair wheels are crooked, wobbly or unstable, then the wheels need to be tightened.

Written By: Kali Slaughter



INTRODUCTION

This guide shows how to tighten the main wheels on the wheelchair.

TOOLS:

- [Socket 19mm](#) (1)
 - [19 mm wrench](#) (1)
-

Step 1 — How to Tighten GEN 2 Wheelchair Wheels



- Inspect the chair to see which wheel is loose.
- Tighten the outer bolt with a 19 mm socket wrench.

Step 2



- Tighten the nut facing the inside of the chair with a 19 mm wrench.

If wheel does not feel properly secured, keep tightening the wheel in small amounts until the wheel is secure.

