



How to fix GEN 2 Wheelchair squeaky parts

If the wheelchair is squeaking, there are many reasons why this may be happening.

Written By: Kali Slaughter



INTRODUCTION

This guide will show how to fix squeaky parts on the wheelchair.

TOOLS:

- [Socket 19mm](#) (1)
 - [10mm Socket](#) (1)
-

Step 1 — How to fix GEN 2 Wheelchair squeaky parts



- Locate the source of the squeaking.
- ⓘ Typically, this will be one of the rear wheels, castor wheels, or brakes.
- Clean and remove any dirt or rust from the source of the squeaking.
- ⓘ You may need to partially disassemble the squeaky part in order to thoroughly clean it.

Step 2



- Check for improper assembly.
- ⓘ Refer to the teardown to check that the wheelchair is properly assembled.
- Check for proper lubrication. If not lubricated, lubricate thoroughly with bike grease.

Step 3



- Roll the wheelchair around to check if the squeaking has stopped.
- ⓘ If the squeaking still continues, recheck the chair until the problem is fixed.

Squeaking can be caused by overly tightened connections. If problem still occurs after going through steps, try to slightly loosen bolts (half a turn of the wrench) near the source of squeaking.